




URINE STRIP 2P


Pack Size
100
Tests

Instant detection of
Glucose + Protein
in urine

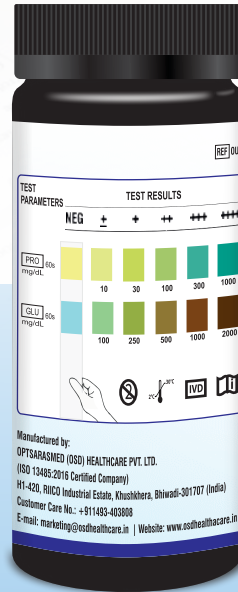


- Easy to use
- Test for 2 parameters
- Safe and sterile
- Accurate
- Fast result


How to use

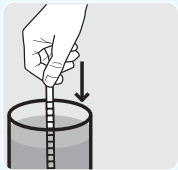

1. Completely immerse reagent areas of the strip in fresh urine container.

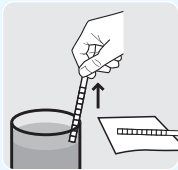

2. Remove immediately and hold the strip sideways for only few seconds.

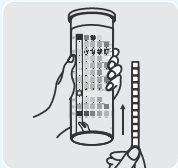

3. Compare reagent areas to corresponding color chart on the bottle label.

Easy to read 2 parameters chart

OmsureTM
URINE STRIP 2P
(Glucose + Protein)
Strips for detection of Glucose and Protein in human Urine

- Store between 2-30°C away from direct sunlight
- Do not touch test area of the strips
- Remove only enough strips for immediate use
- Close cap immediately and tightly

TEST PARAMETERS	TEST RESULTS					
	NEG	±	+	++	+++	++++
PRO mg/dL	[Color]	[Color]	[Color]	[Color]	[Color]	[Color]
GLU mg/dL	[Color]	[Color]	[Color]	[Color]	[Color]	[Color]

Manufactured by:
OPTSARASMED (OSD) HEALTHCARE PVT. LTD.
(ISO 13485:2016 Certified Company)
H1-420, RIICO Industrial Estate, Khushkhhera, Bhiwadi-301707 (India)
Customer Care No.: +911493-403808
E-mail: marketing@osdhealthcare.in | Website: www.osdhealthcare.in

LB-04-00



OPTSARASMED (OSD) HEALTHCARE PVT. LTD.

(ISO 13485:2016 Certified Company)

H1-420, RIICO Industrial Estate, Khushkhhera, Bhiwadi-301707 (India)

Customer Care No.: +911493-403808

E-mail: marketing@osdhealthcare.in | Website: www.osdhealthcare.in